



# NoninConnect™ Model 3230 App User Instructions



NoninConnect 3230 is only compatible with *Bluetooth*® Smart-enabled *Apple*® devices

## Download the NoninConnect app to track and save SpO<sub>2</sub> readings via email


- Simply place the NoninConnect on your finger with your *Bluetooth*® Smart mobile device nearby (up to 10m, 32ft).
- SpO<sub>2</sub> readings will be sent to the *Bluetooth* Smart device and can be emailed as a CSV file or tracked with *Apple*'s Health app.
- The *Apple* devices that are compatible with NoninConnect include: iPhone 4S and newer, iPad 3 and newer, all iPad Air and iPad Minis, and iPod Touch 5th edition and newer. Compatible with iOS 8.1 or newer.
- NoninConnect is **not** compatible with *Apple* devices that do not contain *Bluetooth* Low Energy radios (such as the iPad2) as well as *Microsoft Windows*® and *Google Android*®.

## How to download and use the NoninConnect App

1. Download the free NoninConnect App from the **Apple**® **iTunes**® Store.
2. Insert finger into oximeter.
3. Your mobile *Bluetooth* Smart-enabled *Apple* device will detect the oximeter; select and pair.
4. Accept the disclaimer.

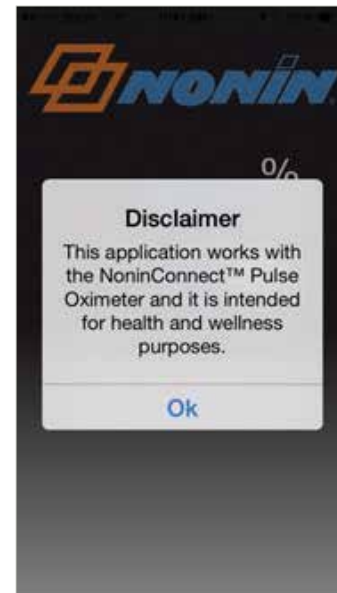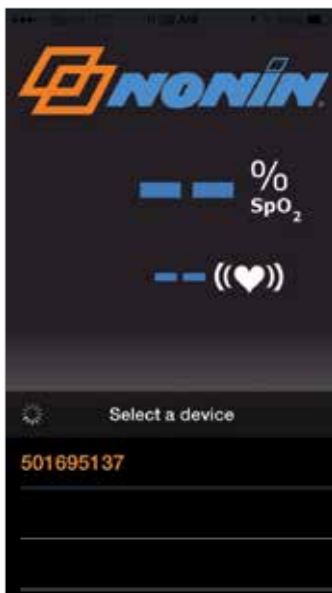

5. View the SpO<sub>2</sub> and pulse rate on the mobile smart device. If necessary, swipe to see both screens.
6. Before removing finger from the oximeter, email the data from the mobile smart device. The data will then be saved in that email.
7. Your Apple mobile smart device will also ask if you would like to save the data in your Apple® Health app. Allow that by clicking on the options and changing them to “on”.

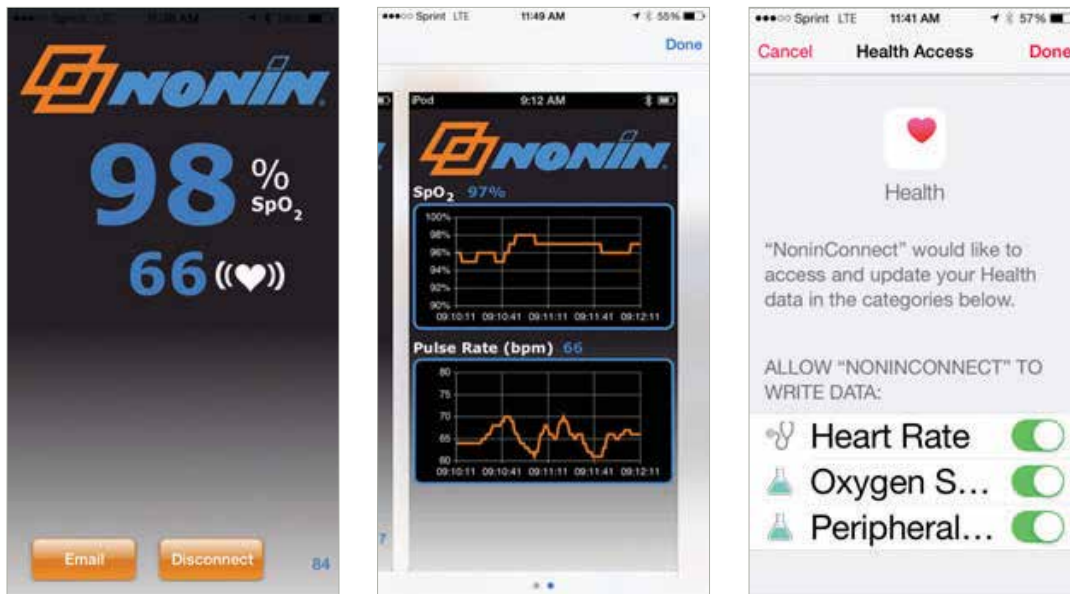

## Customer Support

If you have questions about the NoninConnect App, please contact Nonin Technical Service.

(800) 356-8874 (USA and Canada)

+1 (763) 553-9968 (outside USA and Canada)

Fax: +1 (763) 553-7807

E-mail: [technicalservice@nonin.com](mailto:technicalservice@nonin.com)